ENGLISH

Ophthalmoscopes, Otoscopes/Throat Illuminators (20000, 202XX, 217XX, 250XX Series)

Conforms with: IEC/UL/CSA/EN 60601-1
Les ophthalmoscopes comply with ISO 10942 (Group B)

Symbols	
	Ophthalmoscope Environmental Conditions Temperature limitation Operating: 59°F (10°C)-104°F (49°C) Transport/storage: -4°F (-20°C) - 120°F (55°C)
	Humidity limitation Operating: 30% - 90% Transport/storage: 10% - 95%
	Atmospheric pressure limitation 500 hPa - 1060 hPa
	Attention. Read Operating Manual for Cautions and Instructions for Use
	
	Reorder number
	Product Identifier
	Manufacturer & Date of Manufacture (YYYY-MM-DD)
	Professional use only
	Authorized Representative in the European Community
	Global Trade Item Number
	Batch code

Warnings and Cautions

WARNING The light emitted from the ophthalmoscope is potentially hazardous. The longer the duration of exposure, the greater the risk of ocular damage. Exposure to light from this instrument, when operated at maximum intensity, will exceed the safety guideline after 13 minutes when lit with a LED lamp (blue base) and 23 minutes when lit with a halogen lamp. Exposure times are cumulative for a 24-hour period.

WARNING Lamps may be hot. Lamp should be allowed to cool before removal.

CAUTION Halogen lamps are pressurized to provide maximum efficiency and illumination. Mistreatment may cause shattering. Protect lamp surface against abrasions and scratches. Use protective eyeglasses when handling. Be sure power is off when replacing lamp. Dispose of lamp with care.

Note It is important to note that while the throat illuminator section provides an excellent source of general illumination, more than momentary direct skin contact should be avoided due to possible heat build up.

Repairs

Repairs should be performed at the factory.

Cleaning and Disinfection

For cleaning the mirror or lenses, use only a clean, dry optical-grade cloth. Do not use alcohol or other chemical solutions. The window may be cleaned with a cotton swab or lens paper dipped in alcohol.

The Head housing may be cleaned and disinfected with an appropriate health care low- or intermediate-level cleaner/disinfecting wipe that incorporates either a 1:10 sodium hypochlorite (bleach) solution or isopropyl alcohol as the active disinfection ingredient. Follow wipe manufacturer's instructions for appropriate use, contact times and applicable warnings and precautions. Do not excessively saturate the Head. Do not immerse Head in any solution. Do not sterilize the Head. After disinfection, inspect for visible signs of deterioration. If evidence of damage or deterioration is present, discontinue use and contact Welch Allyn or your area representative.

Assembly
Head attachment (1)
Removing head to use throat illuminator (2)
Lamp replacement (3)

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