Customer Site Requirements - Daytona (P200T)

The following information outlines the equipment customers must have in place before an Optos device can be installed. This document can also be used if a customer wants to upgrade their own Viewing PCs.

More details can be found in the Technical Data Specification. Please contact Optos for more information (see www.optos.com).

Network

Customers must provide the network needed to connect the scan head to the PCs and servers in the system.

Optos will supply expertise on the device and assist a customer network specialist in providing information so the device can be connected to a customer network. Optos is not specialized in networks and therefore can only help support networks from a product data perspective.

Network:	Gigabit Ethernet ¹
Cabling:	CAT 6
Protocol:	TCP/IP
PC Cards:	1000 autosensing NIC ^{1&2}

Electrical Requirements

Customers must provide a dedicated power supply.

Europe:	200-240V, 50/60Hz, 1.5A
North America:	100-120V, 50/60Hz, 3A

Customer Supplied Viewing PCs

When connected to a network, some system configurations may connect to Customer supplied Viewing PCs. These PCs should meet the following specification. We only support the Operating System listed below. Emulators should not be used.

Any PCs being purchased should exceed the recommended specification to gain the longest life span from the new PC.

Existing customer PCs can be used as Viewing PCs.

Processor:	Intel Pentium-4 (or equivalent) ¹
RAM:	1GB ¹
Graphics Card:	1280 x 1024 with 16.2 million colors
Monitor:	CRT or flat panel ³
Hard Disk:	1GB free ¹
Operating System:	Windows XP

Internet Access

Customers are required to provide broadband internet access prior to the installation of the equipment. This ensures that the system can transfer system and diagnostic logs to Optos. This also allows for potential remote diagnostic repair and software upgrades. Optos will never access patient data information files without customer's permission.

Emailing optomap® Retinal Images

Please note that although optomap® images can be emailed using the Review features the device does not receive incoming emails. Customers need their own email address to receive a response to any images sent.

Printing optomap® Retinal Images

Installation of a printer is the responsibility of the customer. Optos does not provide any support for the performance of that printer. Optos cannot guarantee any minimum performance level nor output quality from any standard printer due to the nature of the printing processes. Printed images should therefore not be used for diagnosis.


We would initially recommend a high quality letter/A4 inkjet printer to give a balance between cost of purchase, cost of consumables and image quality. If printing large numbers of images, you may wish to consider printers with higher capital cost, but lower cost of consumables such as a color laser printer.

Space Requirements (to the nearest unit)

The Image Server PC, where configured, must be positioned more than 3 meters (10 feet) from the scan head.

To allow room to operate the device, the scan head should be placed on a table with adequate space available.

	Table Space Requirement	Scan Head Dimensions
Width:	900 mm (36 inches)	440mm (18 inches)
Depth:	600mm (24 inches)	500mm (20 inches)
Height:	Not applicable	795mm (32 inches)
Weight:	Not applicable	28 kg (62 lbs)
Door access width:	Not applicable	760mm (30 inches)

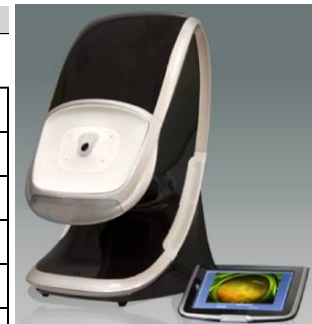Requirements (ambient)

Low ambient light levels are required for effective image capture.

Temperature	Relative Humidity	Atmospheric Pressure
+10°C to +40°C	30% to 75%	700hPa to 1060hPa

Do not use in an environment in which the device may be exposed to flammable anesthetic mixtures.

¹Lower specifications may be used, but with reduced performance and reliability. These levels may be acceptable for features, except 3D Wrap™.

²Network Interface Cards.

³Contrast ratio can vary between different flat screens. It is the customer's responsibility to evaluate the screen image to ensure that it meets his/her requirements for monitoring the retina.

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